

NEW!

Improved! Mini-Max Steam Cleaner

- ✓ SAVE THOUSANDS \$\$\$ IN REPAIRS
- ✓ NEVER TOUCH THE DIAMOND BLADE
- ✓ NO DULLING OF SCISSORS
- ✓ CLEAN OUT CANNULAS
- ✓ CLEANS FINE MICRO-SURGICAL INSTRUMENTS



Faster, easier, safer and more economical way to clean diamond knives and micro-surgical instruments

DGHKOITM
INCORPORATED



CE

DGH-KOI™ INC. INTRODUCES THE NEW KOI 7000 MINI-MAX STEAMER

The unit:

The Mini-Max is compact, lightweight, and portable.

The process:

The Mini-Max's patented system uses an internal heating chamber rather than a pressure tank like conventional steamers. There is no need to fill, drain or worry about pressure build up.

The setup:

It's simple and most importantly safe.

Insert the plastic tube (in back of the unit) into a container of distilled water. Then turn the power actuator on.

In only six minutes the Mini-Max is ready for use for the rest of the day. The foot switch provided allows for hands free operation.

The cleaning:

Hold your knives or instruments under the nozzle, press the foot switch and steam the debris away! Use distilled water to eliminate chemical residue on your knives or instruments.

The warranty:

The Mini-Max comes with a full 1 year warranty on all parts and labor.

The service:

Maintenance, service and repair are available directly from DGH-KOI™ INC.

To order:

Reference the model # KOI 7000.

Call toll free 800-344-9674

Or fax order to 717-582-2406

International ph: 717-582-7749

DGH-KOI™ INC.

Box 120 5055 Spring Rd

Shermans Dale, PA 17090

Email: info@dghkoi.com • www.dghkoi.com